





# **Home Care Technical Specification Grab Bars – Concealed Screw** 8700

#### **Stock Numbers:**

- □ R8712W, DN8712, LR8712W 12" Grab Stainless Steel or White Finish 11/4" Bar
- 8716, R8716W, LR8716W 16" Grab Bar Stainless Steel or White Finish, 1 1/4" Bar
- 8718, L8718, R8718PS, LR8718PS R8718PB, LR8718W, R8718W - 18" Grab Bar

Stainless Steel, Polished Stainless, Polished Brass, or White Finish, 1 1/4" Bar

□ 8724, L8724, R8724PS, LR8724PS, LR8724W R8724PB, R8724W - 24" Grab Bar

Stainless Steel, Polished Stainless, Polished Brass, or White Finish, 1 1/4" Bar

- □ 8730, R8730W 30" Grab Bar Stainless Steel or White, 1 1/4" Bar
- □ 8732, R8732W 32" Grab Bar Stainless Steel or White Finish, 1 1/4" Bar
- R8736PS, R8736W, 8736, L8736-36" Grab Bar

Polished Stainless or White Finish, 1 1/4" Bar

□ 8742, R8742PS, R8742W, R8748W- 42" Grab Bar

> Satin or Polished Stainless Finish, White Finish, 1 1/4" Bar

□ 8748, R8748W - 48" Grab Bar Satin Finish or White Finish, 1 1/4" Bar


### **Materials:**

1 1/4" Grab Bar rods are formed from type 304 stainless steel tubing. 12" - 42" length is 20 gauge 48" length is 18 gauge

## Compliance:

- ADA, ANSI, ASTM, CSA
- Tested to 500 lbs. pull

# **Dimensions:**

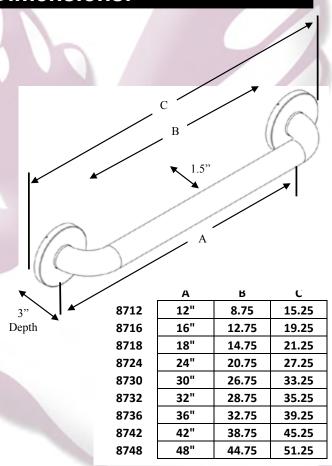

### **Installation Instructions:**

- 1. Position unit on wall in desired location.
- 2. Mark mounting holes.
- 3. Drill starter holes.
- 4. Mount unit on wall using 2" screws (provided).
- 5. Snap flange covers into place.

Note: SecureMount Anchors are recommended for use in nonstud (hollow-wall) applications per installation instructions. (Secure Mount Anchors are sold separately)

Stock Number: 1 1/4" Grab Bars By Creative Specialties

